

# Laryngoscope Handle Components




# Laryngoscope Handle Components- Assembly





# Correct Placement & Fiber Optic System

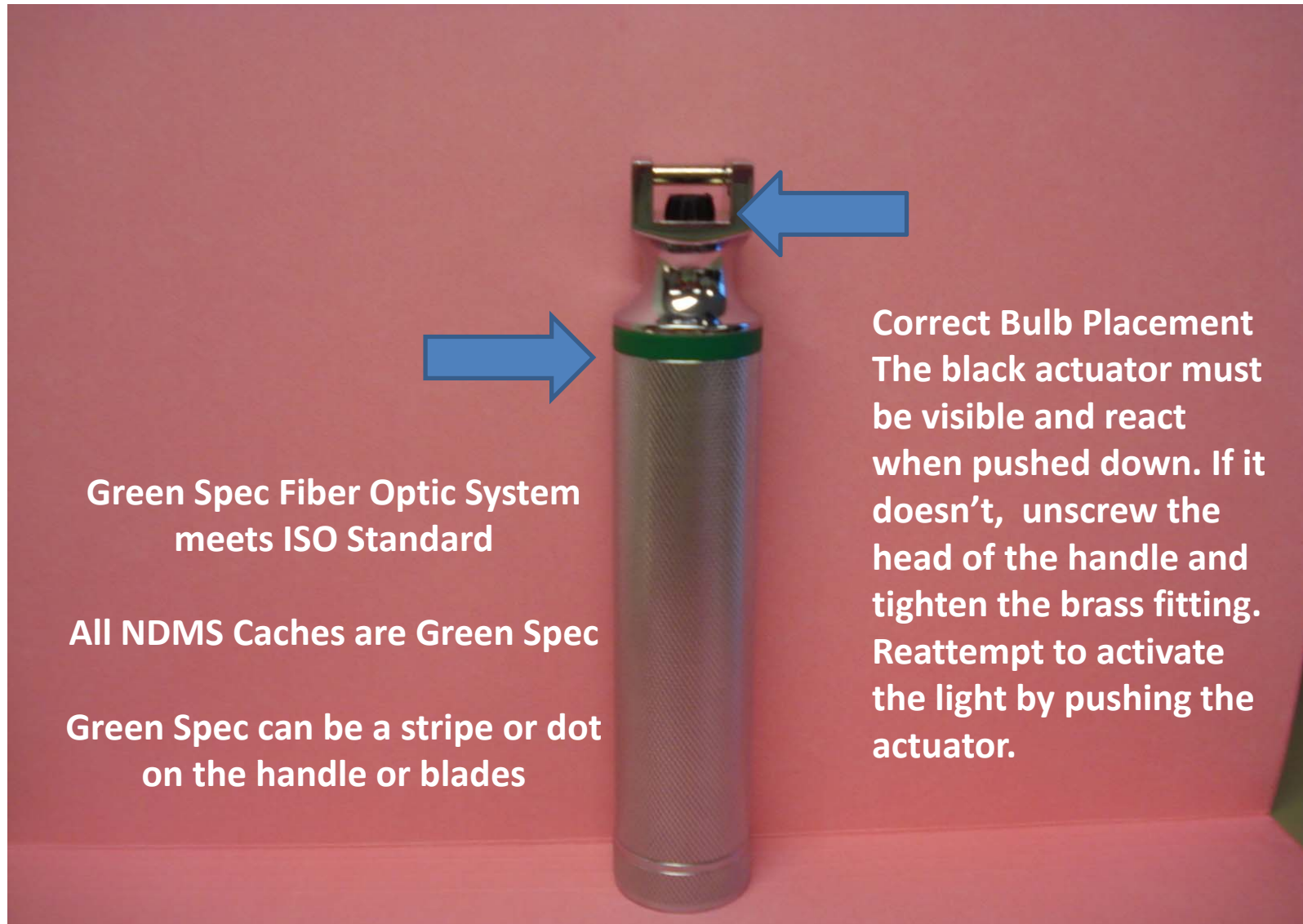

Green Spec Fiber Optic System  
meets ISO Standard

All NDMS Caches are Green Spec

Green Spec can be a stripe or dot  
on the handle or blades

Correct Bulb Placement  
The black actuator must  
be visible and react  
when pushed down. If it  
doesn't, unscrew the  
head of the handle and  
tighten the brass fitting.  
Reattempt to activate  
the light by pushing the  
actuator.

# Quality Assurance

Logistics tests and monitors the laryngoscopes & blades on regular maintenance schedules to ensure the best possible outcome for the mission



Jarring & shifting may occur during the transport process. It is recommended that this equipment be tested prior to use.